


Am I Pregnant?™

Your Pregnancy Test

One Step HCG Pregnancy Test

- ◇ **Over 99% accurate when used from the first day of the expected period.***
- ◇ **Use to test as early as 4 days before your expected per period.**

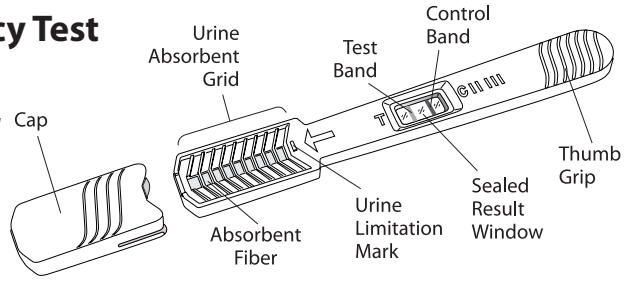

BEFORE YOU BEGIN:



- Read the instructions carefully and have a watch, clock or timer available.
- Do not open the pouch until you are ready to begin the test.
- Check the expiration date on box or pouch. Do not use if it is expired.

HOW TO USE THE TEST

1. The test should be brought to room temperature prior to using, if it was refrigerated.
2. Open the foil pouch immediately before testing. Remove the test stick from pouch. Take off cap.
3. Hold the stick by thumb grip. Point the absorbent grid downward and grid area facing up in urine stream for 5 to 10 seconds until grid area is thoroughly wet.

IMPORTANT: Do not urinate on the window. Take the stick away from urine stream and keep stick in downward position for several seconds to let excess urine drop off naturally. Do not shake away the urine trapped on the grid.

4. You may collect urine in a clean, dry container and **dip only the grid area below the urine limitation mark, in the urine for 5 to 10 seconds.**
5. Replace cap and place the stick on a clean, flat surface with the window facing up. As test is working you will see a light pink stream moving across the window area.

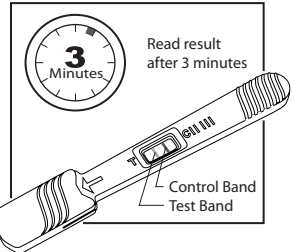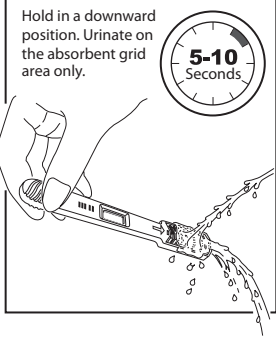

READ YOUR RESULT:

Wait at least 3 minutes to read your result. A positive result may be observed in as early as 1 minute. To verify a negative result, allow the test to run for 5 minutes. Do not interpret result after 10 minutes.

POSITIVE RESULT	NEGATIVE RESULT	INVALID RESULT
<p>One purple band in Control Band (C) area and one purple band in Test Band (T) area indicates a positive result. One band may be fainter or darker than the other.</p>	<p>Only one purple band in Control Band (C) area and no band in Test Band (T) area indicates a negative result.</p>	<p>If no band in C and T area, or only one band in T area, the result is invalid.</p>